



Recommended Monitor Mount Placement

The table and diagram below show recommended locations to place the A-dec Ceiling Monitor Mount. Primary recommendations minimize light and monitor arm interference.

IF THE SYSTEM HAS THIS TYPE OF LIGHT:	INSTALL THE MONITOR MOUNT HERE:	
	Primary Recommendation	Secondary Recommendation
Track-Mounted Ceiling Light	C2*	S1 or S2
Fixed-Mount Ceiling Light	C1	S1 or S2
Chair Mount Light	S1 or S2	C1
Preference-Mounted Light	S1 or S2	C1

* If installing at C2, position the monitor mount a minimum of 6-1/2" (165 mm) in front of the track light (towards 12 o'clock).

* NOTE: C2 position restricts comfortable use of monitor to only supine patient position.

a dec[®]

A-dec Headquarters
2601 Crestview Drive Newberg, Oregon 97132 USA
Tel: 1.800.547.1883 Within USA /Canada
Fax: 1.503.538.0276 Within USA /Canada
Tel: 1.503.538.7478 Outside USA /Canada
www.a-dec.com

**A-dec 311 (Version A)
Chair with Integrated
Floor Box**

Copyright 2017 A-dec Inc.
All rights reserved.

86.0912.01 Rev E

A-dec 311 Plumbing Elevations

5/8" (16 mm) O.D. tubing for wet systems.
40 mm O.D. tubing for dry/semi-dry systems
Central vacuum only. (see general utility notes)

1-1/2" (38 mm) nominal pipe for gravity drain (when required)

2" (51 mm)

1" (25 mm)

5" (127 mm) max. height

1/2" (13 mm) nominal pipe compression fitting with right angle manual shut-off valves for air & water.

1/2" (13 mm) conduit for electrical. (supplied by contractor; check amperage requirements)

PLEASE NOTE: **This drawing is for reference purposes ONLY. It is NOT TO SCALE.**

A-dec Track Light

CAUTION:
Please see note below.

7-1/2" (191 mm)

77-1/2" (1969 mm)

58" (1473 mm)

27-3/4" - 31-1/2" (705 mm-800 mm)

15"-39" (381 mm-991 mm)

7-1/2" (191 mm) from track light to centerline of mounting post. Positioning ensures adequate space for both mounting plate and mounting plate cover.

25" (635 mm)

A-dec 577L Light

A-dec 6300 Light

NOTE: If the track opposite the transformer is closer than 9-1/2" (241 mm) from a wall, the trolley and post assembly must be installed before the track is installed on the ceiling.